


ClassicStar® NIV Full-Face Mask

Premium comfort with anatomical fit and real seal.

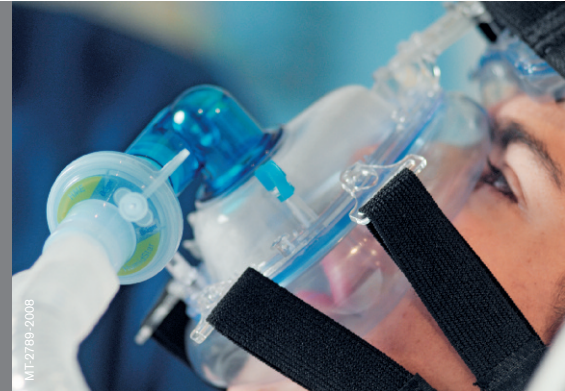

ClassicStar NIV full-face mask is designed for non-invasive ventilation therapy. Because the mask covers the nose and mouth, it provides effective therapy, even in case of mouth breathing.

The ClassicStar NIV full-face mask is for single patient use.

The Dräger Sizing Gauge helps you to choose the right mask size - there is no need to open the mask package.

- The ClassicStar NIV full-face mask has an air-filled mask cushion, which can be adjusted individually to the patient's face. The mask cushion can be inflated or deflated by means of a pump ball. Due to its adaptability, the ClassicStar mask provides increased comfort and effective seal.
- The ClassicStar mask cushion allocates pressure over a larger surface area, which helps to avoid skin irritation.

- The forehead support distributes pressure from the bridge of the nose. The forehead pad and forehead support are adjustable to match the patient's forehead curve.
- The 6-point headgear provides a stable and secure fixation, thus reducing mask displacement and pressure. Safety release is integrated into the headgear for quick removal of the mask.
- Optional fixation points on the mask frame allow for the choice of the most convenient and stable headgear fastening position for each patient.
- The ClassicStar swivel is moveable at 360°. In addition, the swivel can be moved vertically (SE version only), providing more flexibility.



ClassicStar® NIV Full-Face Mask with Standard Elbow (SE).




ClassicStar® NIV Full-Face Mask with Anti-Asphyxia Valve (AAV)



Two-sided pump ball for air-management.

ClassicStar® SE (with Standard Elbow)

The blue elbow indicates that the mask has neither an Anti-Asphyxia Valve nor exhalation ports.

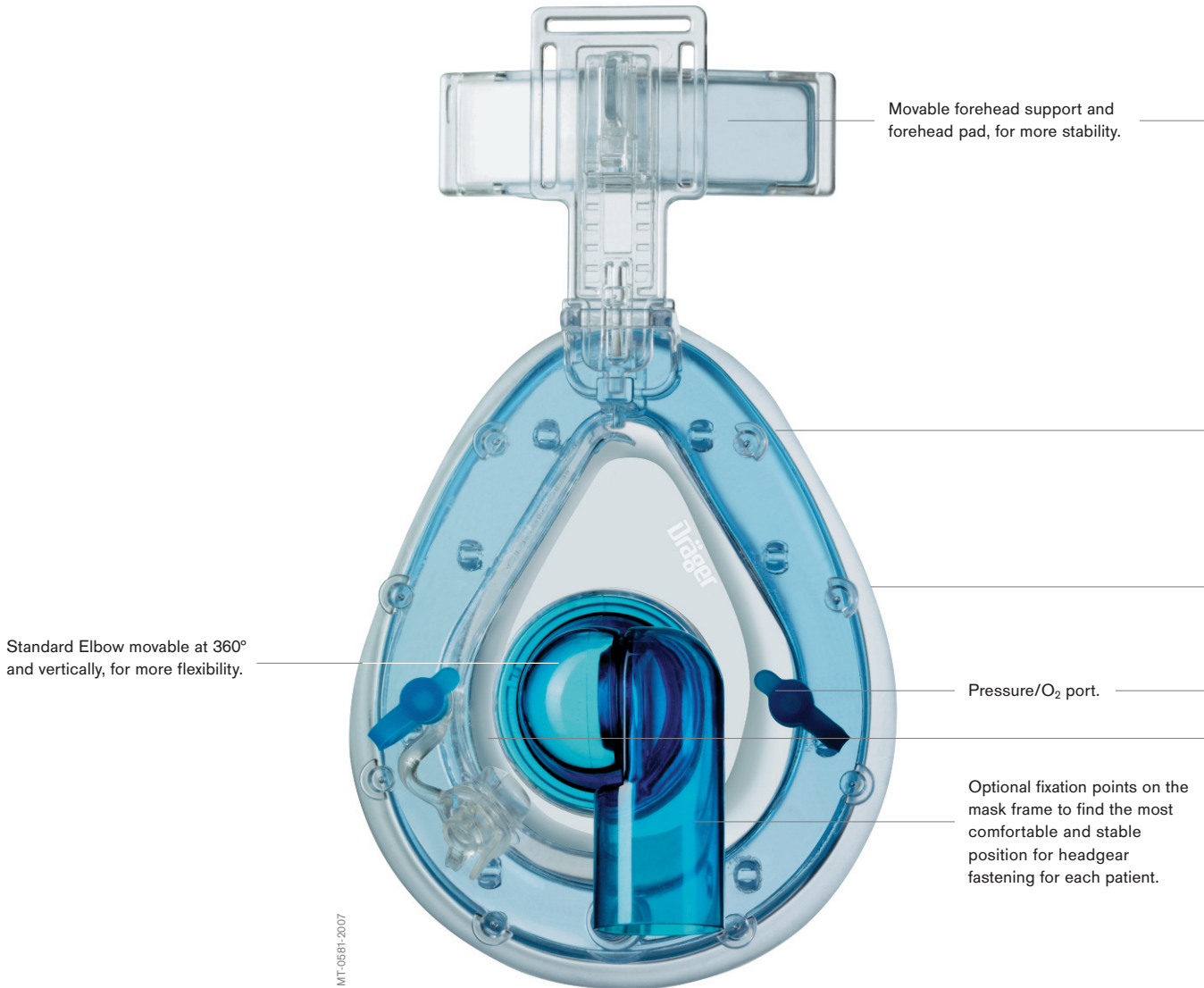


Individual adaptation (formability) of the ClassicStar® NIV full-face mask



ClassicStar® AAV (with Anti-Asphyxia Valve)

The white transparent elbow includes an Anti-Asphyxia Valve that ensures convenient access to room air in the event of reduced pressure (< 3 cm H₂O). Exhalation ports support CO₂ elimination during non-invasive ventilation.

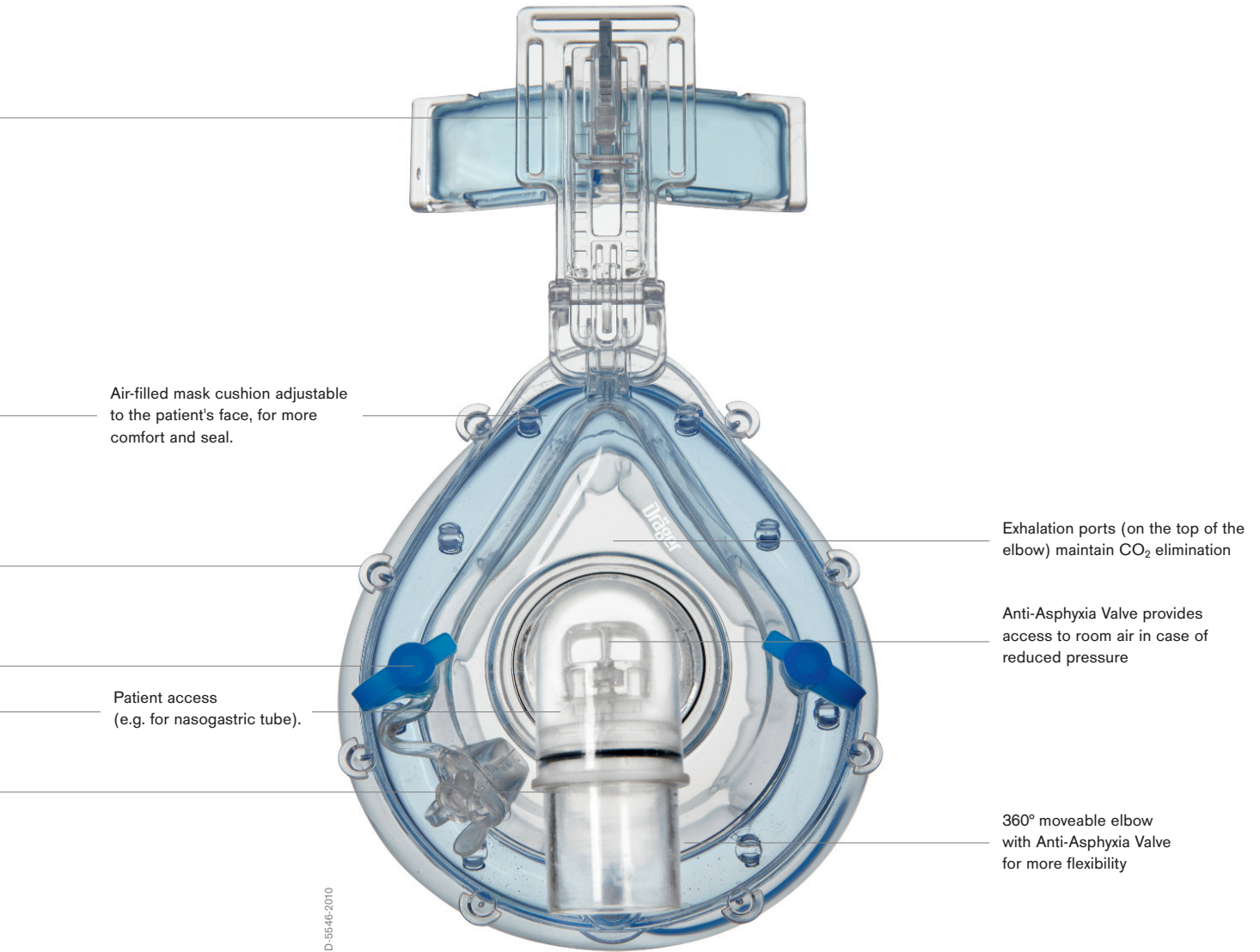

The mask can be adjusted to the patient's face by increasing or decreasing the amount of air in the mask cushion.

ORDER INFORMATION

ClassicStar SE	Packing unit	Order number
ClassicStar SE, full-face mask for non-invasive ventilation with Standard Elbow, size S	1 pc.	MP01573
ClassicStar SE, full-face mask for non-invasive ventilation with Standard Elbow, size M	1 pc.	MP01574
ClassicStar SE, full-face mask for non-invasive ventilation with Standard Elbow, size L	1 pc.	MP01575
ClassicStar AAV		
ClassicStar AAV, full-face mask for non-invasive ventilation with Anti-Asphyxia Valve, size S	1 pc.	MP01570
ClassicStar AAV, full-face mask for non-invasive ventilation with Anti-Asphyxia Valve, size M	1 pc.	MP01571
ClassicStar AAV, full-face mask for non-invasive ventilation with Anti-Asphyxia Valve, size L	1 pc.	MP01572
Two-sided pump ball for air-managment	1 pc.	MP01590
Sizing Gauge for ClassicStar SE and AAV masks for non-invasive ventilation	3 pcs.	MP01592

**HEADQUARTERS**

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany

www.draeger.com

**REGION EUROPE CENTRAL
AND EUROPE NORTH**

Dräger Medical GmbH
Moislinger Allee 53–55
23558 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION EUROPE SOUTH

Dräger Médical S.A.S.
Parc de Haute
Technologie d'Antony 2
25, rue Georges Besse
92182 Antony Cedex, France
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
d1mfr-contact@draeger.com

**REGION MIDDLE EAST, AFRICA,
CENTRAL AND SOUTH AMERICA**

Dräger Medical GmbH
Branch Office Dubai
Dubai Healthcare City, P.O. Box 505108
Dubai, United Arab Emirates
Tel + 971 436 24 762
Fax + 971 436 24 761
contactuae@draeger.com

REGION ASIA / PACIFIC

Dräger Medical
South East Asia Pte Ltd
25 International Business Park
#04-27/29 German Centre
Singapore 609916, Singapore
Tel +65 6572 4388
Fax +65 6572 4399
asia.pacific@draeger.com

Manufacturer:

Dräger Medical GmbH
23542 Lübeck, Germany
The quality management system at
Dräger Medical GmbH is certified
according to ISO 13485, ISO 9001
and Annex II.3 of Directive 93/42/EEC
(Medical devices).